


RESMED MASK FITTING GUIDE

ULTRA MIRAGE™ MASK

ResMed's *Ultra Mirage* Mask provides superior comfort and fit for use with nasal continuous positive airway pressure (CPAP) and bilevel systems. Follow the steps below to ensure the mask fits comfortably without air leaks.

Before fitting the *Ultra Mirage* Mask

- Detach the Quick Release Swivel and attach it to the air tubing, pushing the tubing flush with the ridge on the Quick Release Swivel.
- Ensure the vent cover is properly attached; pinch firmly on either side of the vent cover and push it toward the mask frame.

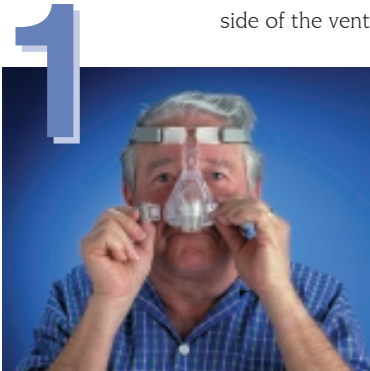

- Place the cushion over your nose.
- Pull the headgear over your head, with the lower straps below your ears.
- Connect the unattached headgear clip to the mask.
- Tighten the Velcro® straps enough that the mask rests in place but is not tight.

- Adjust the forehead support by pressing the tab. Move the tab up to tilt the pads backward and down to tilt the pads forward.
- Reposition the mask by pulling it away from your face and then gently settling it back in place.
- Use the Velcro straps to adjust the fit, pulling adjacent straps simultaneously and evenly.



- Attach the Quick Release Swivel to the mask and the other end of the air tubing to the flow generator.
- After turning the flow generator on and lying down, check that the mask fits comfortably and has no air leaks. If you find leaks, reposition the mask.
- To remove the mask simply detach the headgear clips and pull the mask and headgear away from your head.

