

PHILIPS

DreamStation Go

Portable PAP therapy system



Give your patients powerfully convenient features

Powerful, portable performance

At half the size of our System One and DreamStation platforms, DreamStation Go offers the same therapy and comfort features used by over 5 million PAP users.

Now with heated humidification

DreamStation Go's heated humidifier has been designed to use tap, bottled or distilled water¹ so travelers don't have the inconvenience of packing or purchasing distilled water when they arrive at their destination. It features water-saving technology designed to provide a full night of heated humidification² by actively monitoring ambient conditions and therapy settings and automatically adjusting moisture delivery when humidifier target time is enabled. Also, in the event your patient underfills or forgets to fill the humidifier, our heater plate shutdown feature will take over to prevent dry, uncomfortable air from being delivered to the patient for the remainder of the night.

Patient preferred tubing³

Our 12 mm micro-flexible tubing is designed to provide your patients with added freedom of movement while they sleep. Engineered to minimize bulk and improve mobility, our 12 mm tubing is 58% smaller than our 22 mm performance tubing. It offers enhanced flexibility, portability and connectivity. Click-connections on each end of the micro-flexible tubing make connecting and disconnecting the device and mask simple.

Innovation for patients on the go

DreamStation Go PAP provides the portability and packability travelers want and the effective therapy they need.

Convenient features and benefits

- Gives patients the freedom to use the mask they want⁴
- Intuitive color touchscreen offers simple setup and easy navigation
- Built-in power supply reduces parts managed by patients
- USB port for easy charging of mobile devices
- Complies with FAA requirements for in-flight use⁵

Proven Flex technology

Our clinically proven Flex technologies enhance traditional PAP therapy and are an excellent way to improve patient comfort. DreamStation Go utilizes the same Flex technology as our DreamStation and System One PAP devices.

DreamMapper cross-device connectivity

Users now have the ability to track and monitor their combined progress across both home and portable PAP devices in one convenient dashboard on their smartphone, tablet or computer.

¹In very hard water areas, it is recommended that you use distilled or bottled water.

²Adjusts humidification output when water-saver target time is enabled. Refer to the User Manual for more information.

³Head-to-Head trial of DreamStation 15 mm tubing vs. DreamStation 12 mm micro-flexible tubing, 2016.

⁴Masks with standard 22 mm outer diameter connection.

⁵RTCA/DO-160G section 21, category M.

DreamStation Go accessories create opportunities for you



Heated humidifier

The thinnest PAP heated humidifier among leading PAP brands¹ adds moisture to delivered air with the goal of improving user comfort. Conveniently uses tap, bottled or distilled water.²



Overnight battery

Compact and lightweight with a recharge time of just 4-5 hours.



DreamStation Go small and medium travel kits

Both kits include a protective case and washable bag for the mask and tube. The medium protective case can accommodate DreamStation Go with either the heated humidifier or battery.



DreamStation Go Auto specifications

| | |
|--------------------------|---|
| Pressure range | 4 to 20 cm H ₂ O |
| Pressure mode(s) | CPAP, APAP |
| Flex pressure relief | 1 to 3 |
| Flex modes | C-Flex, C-Flex+, A-Flex |
| Ramp time | 0 to 45 min (5-min increments) |
| Starting ramp pressure | Patient adjustable, SmartRamp |
| Dimensions | 15 cm L x 15 cm W x 5.8 cm H |
| Weight with power supply | 844 g (1.86 lb) |
| Filters | Reusable pollen, disposable fine |
| Device setup | Touchscreen color display |
| Data storage capacity | On-board: > 1 year MicroSD card (optional): lifetime |
| Connectivity | Bluetooth |
| Altitude compensation | Automatic |
| Electrical requirements | 100-240 VAC, 50/60 Hz |
| Warranty | Two years |

DreamStation Go CPAP specifications

| | |
|------------------|-----------------|
| Pressure mode(s) | CPAP |
| Flex modes | C-Flex, C-Flex+ |

*All other specifications are the same as DreamStation Go Auto CPAP

Heated humidifier specifications

| | |
|----------------------|--|
| Dimensions | 14.7 cm L x 15.1 cm W x 5.9 cm H |
| Weight of humidifier | 344.7 g (0.76 lb) |
| Warranty | Two years |
| Tubing compatibility | Micro-flexible 12 mm, 15 mm, and 22 mm standard tubing |
| Tank water capacity | 240 ml |

Ordering information

| Flow generators | Part number |
|--|-------------|
| DreamStation Go CPAP | DSG400S11 |
| DreamStation Go Auto CPAP | DSG500S11 |
| Flow generators with heated humidifier | Part number |
| DreamStation Go CPAP with heated humidifier | DSG400H11 |
| DreamStation Go Auto CPAP with heated humidifier | DSG500H11 |
| Humidifier | Part number |
| DreamStation Go heated humidifier | DSGH |
| DreamStation Go humidifier replacement tank | 1140264 |
| Power | Part number |
| DreamStation Go overnight battery | 1133281 |
| Travel kits | Part number |
| DreamStation Go small travel kit ³ | 1133275 |
| DreamStation Go medium travel kit ⁴ | 1133276 |
| Consumables/replacement | Part number |
| DreamStation micro-flexible 12 mm tubing | PR12 |
| Power cord 6 ft white | 1132551 |
| Power cord 10 ft white | 1132552 |
| Beauty panel - front | 1133741 |
| Beauty panel - rear | 1133742 |
| Reusable pollen filter 2-pk | 1133743 |
| Disposable 30-day fine filter 6-pk | 1133744 |

¹When compared against Transcend, HDM Z1, AirMini (does not offer heated humidifier), Devilbiss

²In very hard water areas, it is recommended that you use distilled or bottled water

³Small travel kit includes an easy-to-pack protective case for the DreamStation Go and a separate washable mask and tube bag.

⁴Medium travel kit includes a protective case that accommodates the DreamStation Go and overnight battery. Also includes a separate washable mask and tube bag.

